 <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p>	Document Code:	DPOTMH-E-55-P01-S03
	Effective Date:	06-30-2022
	Document Type:	Standard Operating Procedure
	Page Number:	1 of 4
	Department/Section:	Clinical Chemistry
	Document Title:	<b>AMYLASE ASSAY</b>

### **PURPOSE:**

To describe in detail how to prepare and process the Amylase Assay test correctly and always in the same manner. Albumin evaluates an individual's hepatic function. This test is used to detect and monitor the clinical course of pancreatitis which is frequently ordered when a patient presents with acute abdominal pain.

### **SCOPE:**


Applies to all Clinical Chemistry Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

### **PERSON RESPONSIBLE:**

Doctors, Nurses, Medical Technologists, Pathologists, Patients, Clerk and Receptionist.


### **GENERAL GUIDELINES:**

- 1 No special preparation is necessary.
- 2 If sample show amylase concentration that exceeds the system's reportable (dynamic) range, the Medical Technologist shall follow this procedure:
  - 2.1 Dilute sample with an equal volume of reagent grade water.
  - 2.2 Analyze
  - 2.3 Multiply the results by two (2) to obtain the original sample's amylase concentration.
- 3 Specimens shall be collected in a Red-Top blood collecting tube.
- 4 Impervious gloves and proper protective clothing shall be worn.
- 5 Collect specimen using standard laboratory procedures.
- 6 Specimens collected shall be considered as biohazardous material.

 <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p>	Document Code:	DPOTMH-E-55-P01-S03
	Effective Date:	06-30-2022
	Document Type:	Standard Operating Procedure
	Page Number:	2 of 4
	Department/Section:	Clinical Chemistry
	Document Title:	<b>AMYLASE ASSAY</b>

B.S. Aquino Drive,  
Bacolod City,  
Negros Occidental,  
6100

- 7 Analyze control material at least once per day to verify system performance.
- 8 Remove serum from clots within four (4) hours of collection.
- 9 Prior to blood collection, the Medical Technologist shall check on the wrist band for patient identification or for the policy on two (2) acceptable person identifiers applied such as allowing the patient to state his/her complete name, date of birth, address or the assigned identification number.
- 10 Tubes must be labeled prior to blood extraction and a sufficient amount of blood shall be extracted to ensure that repeated additional examinations could be performed.
- 11 Endorse the blood samples properly to the Medical Technologist on duty in Clinical Chemistry Section.
- 12 Inspect serum specimen for fibrin clots, as it may cause incomplete sampling of the specimen. Allow specimen to clot completely in order to prevent fibrin clots.
- 13 Serum should not be used for ammonia measurements, because ammonia is produced during the clotting process.
- 14 Results are released within 2 to 3 hours

 <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p> <p>B.S. Aquino Drive, Bacolod City, Negros Occidental, 6100</p>	Document Code:	DPOTMH-E-55-P01-S03
	Effective Date:	06-30-2022
	Document Type:	Standard Operating Procedure
	Page Number:	3 of 4
	Department/Section:	Clinical Chemistry
	Document Title:	<b>AMYLASE ASSAY</b>

### **PROCEDURE:**

1. Blood specimens collected in 5 mL red top tubes are checked if properly labeled and then subjected to centrifugation at 3500 rpm for 5 minutes.
2. Specimens are then bar-coded through the LIS and barcode labels are placed properly in the tubes without overlapping the handwritten details written by the phlebotomist.
3. Bar-coded specimens are placed in the analyzers sample racks. The Medical Technologist then press the start or on button of the analyzer to begin analyses.
4. Results are then copied from the LIS and verified by the Medical Technologist.
5. Once verified, results are released to the HIS wherein the nurses from the different nurse's station in the hospital as well as the Releasing Clerks can see and print the results.

### **REFERENCE:**

1. Ortho Clinical Diagnostics Instruction for Use (IFU).






DR. PABLO O. TORRE  
MEMORIAL HOSPITAL

B.S. Aquino Drive,  
Bacolod City,  
Negros Occidental,  
6100

Document Code:	DPOTMH-E-55-P01-S03
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	4 of 4
Department/Section:	Clinical Chemistry
Document Title:	<b>AMYLASE ASSAY</b>


**APPROVAL:**

	Name/Title	Signature	Date
Prepared by:	<b>ALEXIS F. CORDIA JR., RMT</b> Section Head, Clinical Chemistry	<i>Alexis F. Cordia Jr.</i>	07/01/2022
Verified:	<b>MONICA B. VILLANUEVA, RMT, PhD</b> Laboratory Manager	<i>M. Villanueva</i>	07-13-2022
	<b>MELANIE ROSE B. ZERRUDO, MD, FPSP</b> Chair, Department of Pathology	<i>MRB</i>	07-13-2022
Reviewed:	<b>DENNIS C. ESCALONA, MN, FPSQua</b> Quality Assurance Supervisor	<i>D. Escalona</i>	07-13-2022
Recommending Approval:	<b>ROSARIO D. ABARING, MAN, PhD</b> Ancillary Division Officer	<i>R. Abaring</i>	07-13-2022
	<b>FREDERIC IVAN L. TING, MD</b> OIC - Total Quality Division	<i>F. Ting</i>	7/19/22
Approved:	<b>GENESIS GOLDI D. GOLINGAN</b> President and CEO	<i>G. Golingan</i>	9/7/22

 <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p> <p>B.S. Aquino Drive, Bacolod City, Negros Occidental, 6100</p>	Document Code:	DPOTMH-E-55-P01-WI03
	Effective Date:	06-30-2022
	Document Type:	Work Instruction
	Page Number:	1 of 2
	Department/Section:	Clinical Chemistry
	Document Title:	<b>AMYLASE ASSAY</b>

KEY TASKS	PERSON RESPONSIBLE
1. Collects blood specimen in 5 mL red top tubes, labels properly and then subjected to centrifugation at 3500 rpm for 5 minutes.	Medical Technologist
2. Bar codes specimens through the LIS and places properly in the tubes without overlapping the handwritten details written by the phlebotomist.	
3. Places bar-coded specimens in the analyzers sample racks.	
4. Presses the start or on button of the analyzer to begin analyses.	
5. Verifies results.	
6. Releases results after verification to the HIS wherein the nurses from the different nurse's station in the hospital as well as the Releasing Clerks can see and print the results.	




 <p>B.S. Aquino Drive, Bacolod City, Negros Occidental, 6100</p> <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p>	Document Code:	DPOTMH-E-55-P01-WI03
	Effective Date:	06-30-2022
	Document Type:	Work Instruction
	Page Number:	2 of 2
	Department/Section:	Clinical Chemistry
	Document Title:	AMYLASE ASSAY


**APPROVAL:**

	Name/Title	Signature	Date
Prepared by:	<b>ALEXIS F. CORDIA JR., RMT</b> Section Head, Clinical Chemistry	<i>Alexis F. Cordia Jr.</i>	07/01/2022
Verified:	<b>MONICA B. VILLANUEVA, RMT, PhD</b> Laboratory Manager	<i>M. Villanueva</i>	07-13-2022
	<b>MELANIE ROSE B. ZERRUDO, MD, FPSP</b> Chair, Department of Pathology	<i>MRB</i>	07-13-2022
Reviewed:	<b>DENNIS C. ESCALONA, MN, FPSQua</b> Quality Assurance Supervisor	<i>D. Escalona</i>	07-13-2022
Recommending Approval:	<b>ROSARIO D. ABARING, MAN, PhD</b> Ancillary Division Officer	<i>R. Abaring</i>	07.13.2022
	<b>FREDERIC IVAN L. TING, MD</b> OIC - Total Quality Division	<i>F. Ting</i>	7/19/22
Approved:	<b>GENESIS GOLDI D. GOLINGAN</b> President and CEO	<i>G. Golingan</i>	9/10/22

## FLOWCHART

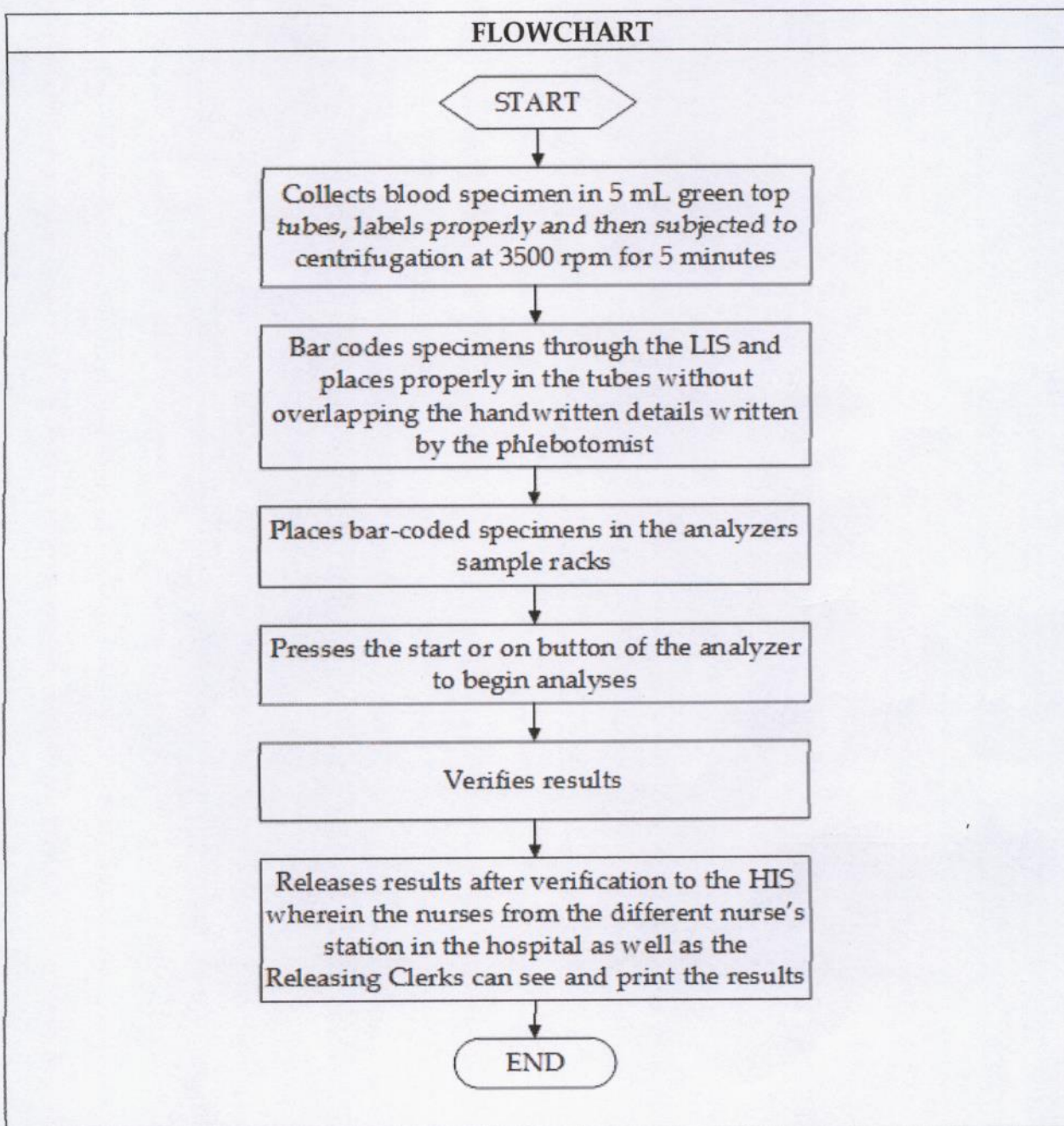



 <p>B.S. Aquino Drive, Bacolod City, Negros Occidental, 6100</p> <p>DR. PABLO O. TORRE MEMORIAL HOSPITAL</p>	Document Code:	DPOTMH-E-55-P01-FC03
	Effective Date:	06-30-2022
	Document Type:	Flowchart
	Page Number:	2 of 2
	Department/Section:	Clinical Chemistry
	Document Title:	AMYLASE ASSAY

**APPROVAL:**

	Name/Title	Signature	Date
Prepared by:	<b>ALEXIS F. CORDIA JR., RMT</b> Section Head, Clinical Chemistry	<i>Alexis F. Cordia Jr.</i>	07/01/2022
Verified:	<b>MONICA B. VILLANUEVA, RMT, PhD</b> Laboratory Manager	<i>M. Villanueva</i>	07.13.2022
	<b>MELANIE ROSE B. ZERRUDO, MD, FPSP</b> Chair, Department of Pathology	<i>MB</i>	07-13-2022
Reviewed:	<b>DENNIS C. ESCALONA, MN, FPSQua</b> Quality Assurance Supervisor	<i>D. Escalona</i>	07-13-2022
Recommending Approval:	<b>ROSARIO D. ABARING, MAN, PhD</b> Ancillary Division Officer	<i>R. D. Abaring</i>	07.13.2022
	<b>FREDERIC IVAN L. TING, MD</b> OIC - Total Quality Division	<i>F. I. L. Ting</i>	7/13/22
Approved:	<b>GENESIS GOLDI D. GOLINGAN</b> President and CEO	<i>Genesis Goldi D. Golingan</i>	9/7/22