

Document Code:	DPOTMH-E-57-P01-S14	
Effective Date:	Date: 06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	1 of 3	
Department/Section:	Hematology	
Document Title:	APT TEST	

#### **PURPOSE:**

To describe in detail how to prepare and process the APT test correctly and always in the same manner. It is used to differentiate if the blood is from the mother or the peopate.

#### SCOPE:

Applies to all Hematology Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

#### PERSON RESPONSIBLE:

Medical Technologists (Medical Laboratory Scientists), Pathologists, Medical Doctors, Nurses, Medical Trainees, Laboratory Clerks

#### PROCEDURE:

- 1. Hand washing shall be performed before and after the procedure.
- 2. The wearing of Personal Protective Equipment such as gloves, masks, goggles/face shield and gowns shall be observed in doing the procedure.
- 3. Sample (gastric aspirate) must be placed in a clear, dry, leak-proof, screw-capped container.
- 4. The Medical Technologist or Nurse forwards properly the sample and patient's request to the Hematology Section. Medical Technologist then checks sample label, volume and clots prior to analysis. Not properly labeled sample is given back to the one endorsed for correct labeling.
- 5. Mix 1 part of gastric aspirate to 9 parts of 2% NAOH.

Interpretation of Result/ manner of reporting: <u>PINK</u>: Fetal origin YELLOW OR BROWN: Maternal origin

- 6. The Medical Technologist notes macroscopic findings in the request and encodes results in the system.
- 7. Specimens collected shall be considered as biohazardous material.



Document Code:	DPOTMH-E-57-P01-S14	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	2 of 3	
Department/Section:	Hematology	
Document Title:	APT TEST	

- The Medical Technologist/Laboratory Clerk prints out results for outpatients at the reception area upon presentation of the Official Receipt (OR) and results for admitted patients can be viewed and printed out by nurses to its respective stations.
- 9. Names and results of patient shall be recorded in the Hematology/Miscellaneous logbook.
- 10. Turn-around time for releasing Apt test result is within 2 hours.

#### **REFERENCE:**

https://en.m.wikipedia.org/wiki/Alkali\_denaturation\_test



Document Code:	DPOTMH-E-57-P01-S14	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	3 of 3	
Department/Section:	Hematology	
Document Title:	APT TEST	

# APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	fluds.	07-06-2022
	Section Head, Hematology	quans	07 00 20
	TIFFANY B. VILLANUEVA-COO, RMT	Blu	7 / 0000
	Laboratory Manager	(CUM	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD	Ari Vanuers	7-6-2022
	Laboratory Manager	Cr. vanura	1-4-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP	Warenofe Generals	7-6-2022
	Chair, Department of Pathology	00	
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua	0	07-02-2022
	Quality Assurance Supervisor		
	ROSARIO D. ABARING, MAN, PhD	(D) 1 1 1	
Recommending	Ancillary Division Officer	Klaing & l.	07.06.2022
Approval:	FREDERIC IVAN L. TING, MD	12	7/8/22
• •	OIC - Total Quality Division		
Approved:	GENESIS GOLDI D. GOLINGAN	Anox	0/1/20
11	President and CEO	700	7/11/22



Document Code:	DPOTMH-E-57-P01-WI14	
Effective Date:	06-30-2022	
Document Type:	Work Instruction	
Page Number:	1 of 2	
Department/Section:	Hematology	
Document Title:	APT TEST	

	KEY TASKS	PERSON RESPONSIBLE
1.	Encodes the request of the patient to BIZBOX system.	Laboratory Clerks and Nurses
2.	Extracts blood sample from the patients.	
3.	Ensures sample is acceptable for testing.	
4.	Forwards properly the sample and patient's request to the Hematology Section.	
5.	Disposes properly and safely biohazardous and infectious wastes and materials.	Medical Technologist/Nurse
6.	Processes and analyzes samples.	
7.	Releases and validates result thru BIZBOX System.	
8.	Records result to Hematology logbooks.	
9.	Endorses the patient's results to the patient or doctor.	Laboratory Clerks and Nurse




Document Code:	DPOTMH-E-57-P01-WI14	
Effective Date:	06-30-2022	
Document Type:	Work Instruction	
Page Number:	2 of 2	
Department/Section:	Hematology	
Document Title:	APT TEST	

## APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	aluds.	07-04-2022
	Section Head, Hematology  TIFFANY B. VILLANUEVA-COO, RMT  Laboratory Manager	Ollw	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	A: lle nuer.	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	blung- Genedo	7-4-2022
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor	2	07-04-2022
Recommending	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer	Alaine to a.	07-06-2022
Approval:	FREDERIC IVAN L. TING, MD OIC - Total Quality Division	3	7/8/22
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO		9/1/22



Document Title:	APT TEST
Department/Section:	Hematology
Page Number:	1 of 2
Document Type:	Flowchart
Effective Date:	06-30-2022
Document Code:	DPOTMH-E-57-P01-FC14

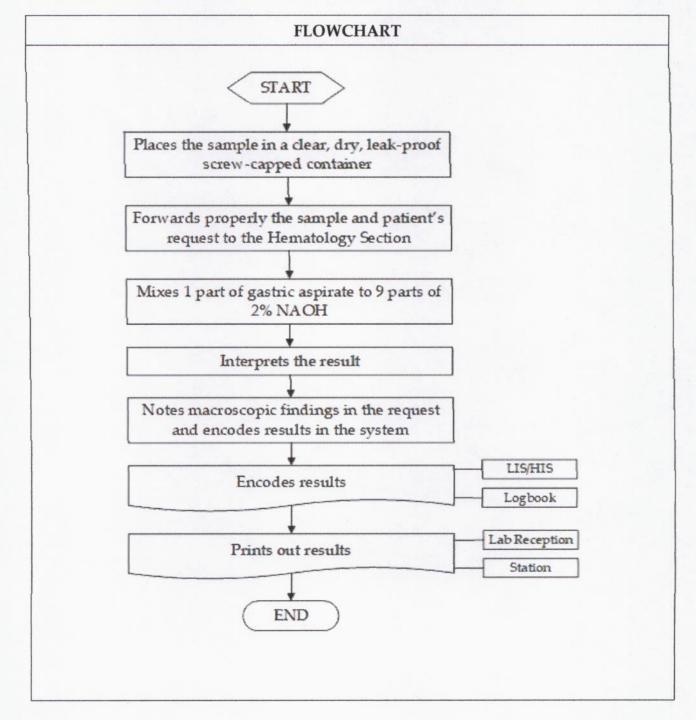



Document Code:	DPOTMH-E-57-P01-FC14
Effective Date:	06-30-2022
Document Type:	Flowchart
Page Number:	2 of 2
Department/Section:	Hematology
Document Title:	APT TEST

### APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	afads.	07-04-2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	<b>D</b> lw	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	A Unner	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	Lauf feut	7-4-202
Reviewed by:	DENNIS C. ESCALONA, MN, FPCHA, FPSQua Quality Assurance Supervisor	Ω	07-04-202
Recommending Approval:	ROSARIO D. ABARING, MN, MBA-HA, PhD, FPCHA Ancillary Services Division Officer FREDERIC IVAN L. TING, MD OIC- Total Quality Division	Plaine b. 4	01.06.202
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO	-	9/11/22