

Document Code: DPOTMH-E-57-P01-S15	
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	1 of 3
Department/Section:	Hematology
Document Title:	CLOT RETRACTION TIME (CRT)

PURPOSE:

To describe in detail how to prepare and process the Clot Retraction Time test correctly and always in the same manner. It is used in measuring coagulation ability, when more sophisticated methods are unavailable and to evaluate family history of blood clotting disorders, excessive bleeding and bruising.

SCOPE:

Applies to all Hematology Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

PERSON RESPONSIBLE:

Medical Technologists (Medical Laboratory Scientists), Pathologists, Medical Doctors, Nurses, Medical Trainees, Laboratory Clerks

PROCEDURE:

- 1. Hand washing shall be performed before and after the procedure.
- The wearing of Personal Protective Equipment such as gloves, masks, goggles/face shield and gowns shall be observed in doing the procedure.
- 3. Requests, unless indicated as "STAT" shall be included in the batch of examinations processed in a first come first serve basis.
- 4. A water bath or incubator must be at 37C set ready for the procedure.
- The Medical Technologist collects 5 mL venous blood and dispenses into a centrifuge tube. Place the tube in 37C water bath or incubator and starts stopwatch.
- 6. Loosen clot gently from the wall of the tube after 10 minutes using thin platinum wire.
- 7. At the end of 1 hour, carefully remove clot from the tube with a wire hook and measure height of the serum.

NOTE: If serum is grossly contaminated with red cells, centrifuge the tube at 1500 rpm for 10 minutes and record height of serum.



Document Code:	DPOTMH-E-57-P01-S15
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	2 of 3
Department/Section:	Hematology
Document Title:	CLOT RETRACTION TIME (CRT)

CALCULATION:

Height of serum (mL) x 100
Height of whole blood (mL)

- 8. Normal value: more than 40%
- 9. The Medical Technologist forwards properly the patient's request with the results and his name initials to the Hematology Section for encoding in the system.
- 10. Results for outpatients are printed at the reception area upon presentation of the Official Receipt (OR) and results for admitted patients are can be viewed and printed out by nurses to its respective stations.
- 11. Critical CRT results must be reported to the Attending Physician or Nurse immediately.
- 12. Names and results of patient shall be recorded in the Hematology logbook.
- 13. Turn-around time for releasing CRT result is within 2 hours.

REFERENCE:

https://www.pathologyoutlines.com/topic/coagulationclotretraction.html



Document Code:	DPOTMH-E-57-P01-S15
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	3 of 3
Department/Section: Hematology	
Document Title:	CLOT RETRACTION TIME (CRT)

APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	dads.	07-06-2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Blu	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager		7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	Brenfe-Generalo	7-6-2022
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor	2	9-06-2022
Recommending	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer	Relaing . f. b.	07.06.2022
Approval:	FREDERIC IVAN L. TING, MD OIC - Total Quality Division	A	7/8/22
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO	#	9/11/22



Document Code:	DPOTMH-E-57-P01-WI15
Effective Date:	06-30-2022
Document Type:	Work Instruction
Page Number:	1 of 2
Department/Section:	Hematology
Document Title:	CLOT RETRACTION TIME (CRT)

	KEY TASKS	PERSON RESPONSIBLE
1.	Encodes the request of the patient to BIZBOX system.	Laboratory Clerks and Nurses
2.	Extracts blood sample from the patients.	
3.	Ensures sample is acceptable for testing.	
4.	Disposes properly and safely biohazardous and infectious wastes and materials.	
5.	Forwards properly the patient's request with the results and his name initials to the Hematology Section for encoding in the system.	
6.	Processes and analyzes samples.	
7.	Releases and validates result thru BIZBOX System.	
8.	Records result to Hematology logbooks.	
9.	Endorses the patient's results to the patient or doctor.	Laboratory Clerks and Nurses




Document Title:	CLOT RETRACTION TIME (CRT)
Department/Section:	Hematology
Page Number:	2 of 2
Document Type:	Work Instruction
Effective Date:	06-30-2022
Document Code: DPOTMH–E-57-P01-WI15	

APPROVAL:

	Name/Title	Signature	Date	
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	dods.	07-06-2022	
	Section Head, Hematology	C		
	TIFFANY B. VILLANUEVA-COO, RMT	村儿	7-6-2022	
	Laboratory Manager	from	, a anala	
Verified:	MONICA B. VILLANUEVA, RMT, PhD	drillanur.	7-6-2026	
	Laboratory Manager			
	MELANIE ROSE B. ZERRUDO, MD, FPSP	1. On Some from So	7-6-2022	
	Chair, Department of Pathology	Jan	/ 6	
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua	2	07-06-2022	
	Quality Assurance Supervisor		- COIL	
	ROSARIO D. ABARING, MAN, PhD	01.11		
Recommending	Ancillary Division Officer	Klains &. V.	07.06. 2022	
Approval:	FREDERIC IVAN L. TING, MD	1	7/8/22	
••	OIC - Total Quality Division	1		
Approved:	GENESIS GOLDI D. GOLINGAN		9/11/22	
• •	President and CEO	1	1111/20	



Document Code:	Document Code: DPOTMH-E-57-P01-FC15	
Effective Date:	06-30-2022	
Document Type:	Flowchart	
Page Number:	1 of 2	
Department/Section:	Hematology	
Document Title:	CLOT RETRACTION TIME (CRT)	

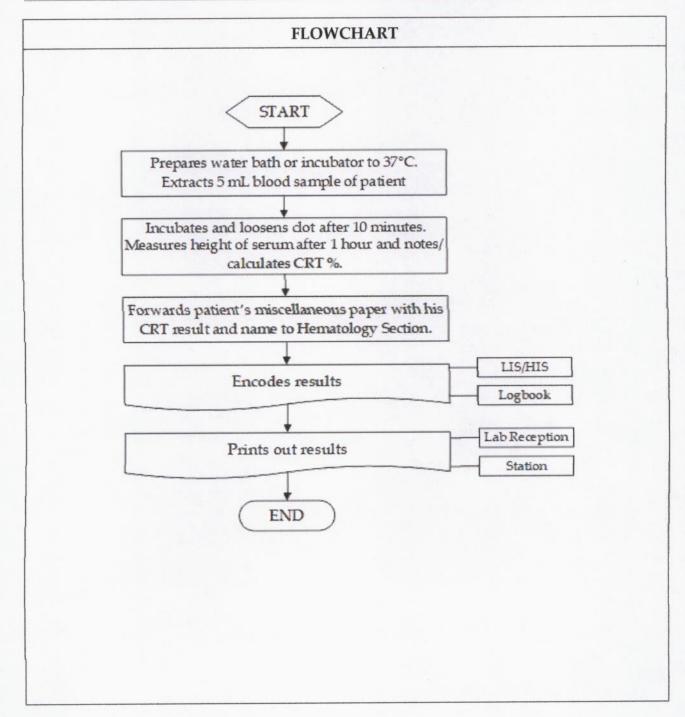



Document Code: DPOTMH–E-57-P01-FC15	
Effective Date:	06-30-2022
Document Type:	Flowchart
Page Number:	2 of 2
Department/Section:	Hematology
Document Title:	CLOT RETRACTION TIME (CRT)

APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	ofods.	07-06-2022
	Section Head, Hematology TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Blo	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	Ar. Vanuer	- 7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	alweigh feur	7-6-202
Reviewed by:	DENNIS C. ESCALONA, MN, FPCHA, FPSQua Quality Assurance Supervisor	0	07-06-2022
Recommending	ROSARIO D. ABARING, MN, MBA-HA, PhD, FPCHA Ancillary Services Division Officer	Relains & L	07.06.202
Approval:	FREDERIC IVAN L. TING, MD OIC- Total Quality Division	#	7/8/2
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO		9/11/22