

Document Code:	DPOTMH-E-57-P01-WI28	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	1 of 3	
Department/Section:	Hematology	
Document Title: EXUDATE/TRANSUDATE - CELL COUNT & DIFFERENTIAL COUNT		

PURPOSE:

To describe in detail how to process the Exudate/Transudate Cell and Differential Count test correctly and always in the same manner. Cell count and differential count are performed to determine the predominant cell type present in the fluid, which can suggest certain diseases.

SCOPE:

Applies to all Hematology Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

PERSON RESPONSIBLE:

Medical Technologists (Medical Laboratory Scientists), Pathologists, Medical Doctors, Nurses, Medical Trainees, Laboratory Clerks

GENERAL GUIDELINES:

- Body fluid sample must be placed separately in a clear, dry, leak-proof, sterile vials container labeled as number 1 for Chemistry and Serology; number 2 for Microbiological studies; and number 3 for Hematology cell counts and transported promptly to Laboratory.
- 2. Exudate/transudate cell count and differential count results shall be released and observe the Releasing of Results Policy. Turn-around time for releasing exudate/transudate cell count and differential count result is within 3 hours.

PROCEDURE:

- The Medical Technologist or Nurse forwards properly the sample and patient's request to the Hematology Section. Medical Technologist then checks sample label, volume and clots prior to analysis. Not properly labeled fluid sample or with coagulum is given or called back to the one endorsed for correct labeling or for another sample submission.
- 2. The Medical Technologist examines and records the gross appearance of the body fluid. Describes the color, transparency, reactions and approximate volume.



Document Code:	DPOTMH-E-57-P01-WI28	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	2 of 3	
Department/Section:	Hematology	
Document Title: EXUDATE/TRANSUDATE – CELL COUNT & DIFFERENTIAL COUNT		

- 3. Swirl the vial to mix the fluid and load (diluted or undiluted) into the counting chamber. Let it stand for a few minutes.
- Count the red and white blood cells. Note the percentage of normal and crenated red blood cells.
- 5. Make a wet mount smear, air dry and stain using Wright's stain for the differential count.
- 6. The Medical Technologist notes macroscopic and microscopic findings in the request and encodes results in the system.
- 7. The Medical Technologist/Laboratory Clerk prints out results for outpatients at the reception area upon presentation of the Official Receipt (OR) and results for admitted patients can be viewed and printed out by nurses to its respective stations.
- 8. Names and results of patient shall be recorded in the Hematology/Miscellaneous logbook.

REFERENCE:

- 1. www.gundersenhealth.org Body Fluid Analysis
- 2. Davidsohn-Henry, Clinical Diagnosis by Laboratory Methods, 15th Edition



Document Code:	DPOTMH-E-57-P01-WI28
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	3 of 3
Department/Section:	Hematology
Document Title:	EXUDATE/TRANSUDATE – CELL COUNT & DIFFERENTIAL COUNT

APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	ofues.	07-04-2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Blu	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	Atillanuer.	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	samp - funds	7-6- 2022
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor	2	04-04-20 22
Recommending Approval:	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer FREDERIC IVAN L. TING, MD OIC - Total Quality Division	Plaing & l.	7/8/22
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO	#3	9/11/22



Document Code:	DPOTMH-E-57-P01-WI28	
Effective Date:	06-30-2022	
Document Type:	Work Instruction	
Page Number:	1 of 2	
Department/Section:	Hematology	
Document Title:	EXUDATE/TRANSUDATE – CELL COUNT & DIFFERENTIAL COUNT	

KEY TASKS		PERSON RESPONSIBLE
1.	Forwards properly the patient's request to the Hematology Section.	Laboratory Clerks and Nurses
2.	Ensures sample is acceptable for testing.	
3.	Properly and safely dispose of biohazardous and infectious wastes and materials.	
4.	Process and analyze samples. Report result with significant findings.	
5.	Processes and analyzes samples.	
6.	Releases and validates result thru BIZBOX System.	
7.	Records result to Hematology logbooks.	
8.	Endorses the patient's results to the patient or doctor.	Laboratory Clerks and Nurses




Document Code:	DPOTMH-E-57-P01-WI28	
Effective Date:	06-30-2022	
Document Type:	Work Instruction	
Page Number:	2 of 2	
Department/Section:	Hematology	
Document Title: EXUDATE/TRANSUDATE - CELL COUNT & DIFFERENTIAL COUNT		

APPROVAL:

Name/Title	Signature	Date
REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	afads.	07-04-2022
TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Blu	7-6-2022
MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	Atillanur	7-6-2022
MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	Lang gunds	7-6-2002
DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor	٥	04-02-2022
ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer FREDERIC IVAN L. TING, MD	Plainie t. J.	07.06.2023
OIC - Total Quality Division GENESIS GOLDI D. GOLINGAN President and CEO		9/11/22
	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer FREDERIC IVAN L. TING, MD OIC - Total Quality Division	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer FREDERIC IVAN L. TING, MD OIC - Total Quality Division



Document Code:	DPOTMH-E-57-P01-FC28	
Effective Date:	06-30-2022	
Document Type:	Flowchart	
Page Number:	1 of 2	
Department/Section:	Hematology	
Document Title: EXUDATE/TRANSUDATE – CELL COUNT & DIFFERENTIAL COUNT		

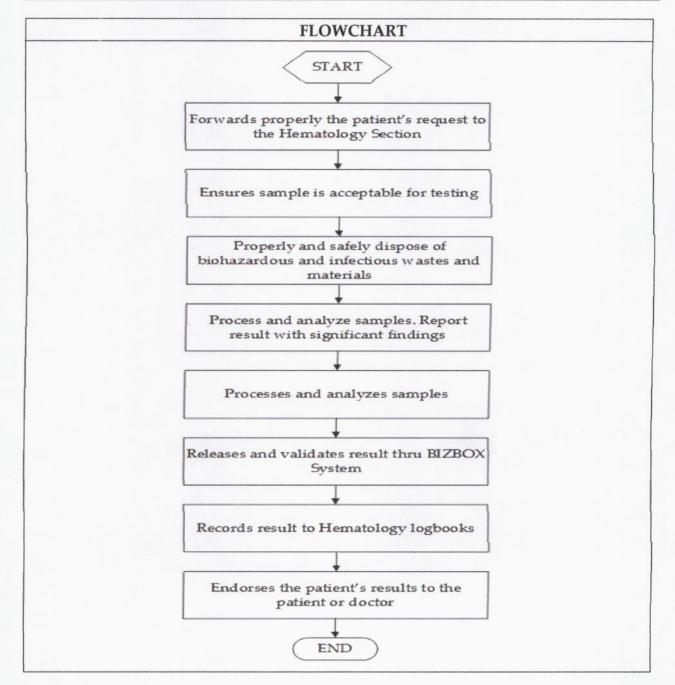



Document Code:	DPOTMH-E-57-P01-FC28	
Effective Date:	06-30-2022	
Document Type:	Flowchart	
Page Number:	2 of 2	
Department/Section:	Hematology	
Document Title: EXUDATE/TRANSUDATE – CELL COUNT & DIFFERENTIAL COUN		

APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	14.	
	Section Head, Hematology	Lads.	07-06-2022
	TIFFANY B. VILLANUEVA-COO, RMT	相	1-6-2022
	Laboratory Manager	pu	10 2012
V7: C: - J.	MONICA B. VILLANUEVA, RMT, PhD	Arillanue	7 / 20
Verified:	Laboratory Manager	arilla mui	- 1-6-NU
	MELANIE ROSE B. ZERRUDO, MD, FPSP	10.m. 2 1	7-6-20
	Chief, Department of Pathology	A Sur	16.00
Reviewed:	DENNIS C. ESCALONA, MN, FPCHA, FPSQua	a	07-04-2022
	Quality Assurance Supervisor		01-04-6022
	ROSARIO D. ABARING, MAN, PhD	01.	1
Recommending	Ancillary Division Officer	Helaing &.	0.07-06.2
Approval:	FREDERIC IVAN L. TING, MD	Klaime st.	7/1/20
	OIC-Total Quality Division	1	1000-
Approved:	GENESIS GOLDI D. GOLINGAN	Alon	211
	President and CEO	100	-9/4/22