

Document Code:	DPOTMH-E-57-P01-S25	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	1 of 3	
Department/Section:	Hematology	
Document Title:	TZANCK TEST	

### **PURPOSE:**

To establish guidelines in collecting sample for Tzanck test for Laboratory diagnostic procedure and analysis. It mainly used in an acute setting to rapidly detect a herpes infection or to distinguish Stevens-Johnson syndrome/ toxic epidermal necrolysis (SJS/TEN) from staphylococcal scalded skin syndrome. It can be used to diagnose a variety of cutaneous infections and blistering diseases.

## SCOPE:

Applies to all Hematology Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

### PERSON RESPONSIBLE:

Medical Technologists (Medical Laboratory Scientists), Pathologists, Medical Doctors, Nurses, Medical Trainees, Laboratory Clerks

#### **GENERAL GUIDELINES:**

- 1. Positive Tzanck Smear results must be reported to the Attending Physician or Nurse immediately.
- 2. Tzanck Smear results shall be released and observe the Releasing of Results Policy.
- 3. Turn-around time for releasing Tzanck Smear result is within 2 hours.

#### PROCEDURE:

- 1. The Medical Technologist forwards properly the patient's request to the Hematology Section. Medical Technologist then checks and selects a fresh vesicle, blister or pustule or directly from the crusted lesion of the patient.
- 2. Using a blunt scalpel blade, the Medical Technologist gently unroof the lesion.
- 3. Scrapes the base of the lesion.
- 4. Smears the tissue onto a clean microscope slide.



Document Code:	DPOTMH-E-57-P01-S25	
Effective Date:	06-30-2022	
Document Type:	Standard Operating Procedure	
Page Number:	2 of 3	
Department/Section:	Hematology	
Document Title:	TZANCK TEST	

- 5. Allow it to dry in the air.
- 6. Fix and stain the specimen.
- 7. The Medical Technologist notes microscopic findings and encodes results in the system.

#### MANNER OF REPORTING:

If negative: NO MULTINUCLEATED GIANT CELL SEEN ON SMEAR

If positive: POSITIVE FOR MULTINUCLEATED GIANT CELLS

- 8. The Medical Technologist/Laboratory Clerk prints out results for outpatients at the reception area upon presentation of the Official Receipt (OR) and results for admitted patients can be viewed and printed out by nurses to its respective stations.
- 9. Names and results of patient shall be recorded in the Hematology/Miscellaneous logbook.

#### REFERENCE:

https://en.m.wikipedia.org/wiki/Tzanck test



Document Code:	DPOTMH-E-57-P01-S25
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	3 of 3
Department/Section:	Hematology
Document Title:	TZANCK TEST

# APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	dads.	07-04-2022
	Section Head, Hematology  TIFFANY B. VILLANUEVA-COO, RMT  Laboratory Manager	Film	1-6 - 2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD	A) illenura	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP	Dung - Guedo	7-6-2002
Reviewed:	Chair, Department of Pathology  DENNIS C. ESCALONA, MN, FPSQua	0	
Reviewed.	Quality Assurance Supervisor		07-06-2022
Recommending	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer	Belaving & b.	07-06-2022
Approval:	FREDERIC IVAN L. TING, MD OIC - Total Quality Division	13	7/8tm
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO		9/11/22



Document Code:	DPOTMH-E-57-P01-WI25
Effective Date:	06-30-2022
Document Type:	Work Instruction
Page Number:	1 of 2
Department/Section:	Hematology
Document Title:	TZANCK TEST

	KEY TASKS	PERSON RESPONSIBLE
Forwards properly the patient's request to the Hematology Section.  Laboratory Clerk		Laboratory Clerks and Nurses
2.	Checks and selects a fresh vesicle, blister or pustule or directly from the crusted lesion of the patient.	
3.	Processes and analyzes samples.	Medical Technologist/Nurse
4.	Releases and validates result thru BIZBOX System.	Medical Technologist/Nurse
5.	Records result to Hematology logbooks.	
6.	Endorses the patient's results to the patient or doctor.	Laboratory Clerks and Nurses




Document Code:	DPOTMH-E-57-P01-WI25
Effective Date:	06-30-2022
Document Type:	Work Instruction
Page Number:	2 of 2
Department/Section:	Hematology
Document Title:	TZANCK TEST

## **APPROVAL:**

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	dads.	07-06- 2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	78Vw	1-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	di Unuen	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	Lhung - Jewels	7-6-20-32
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua Quality Assurance Supervisor	2	07-02-2022
Recommending Approval:	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer FREDERIC IVAN L. TING, MD OIC - Total Quality Division	Maing.k. 1.	07.06.2022
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO		9/11/22



Document Code:	DPOTMH-E-57-P01-FC25
Effective Date:	06-30-2022
Document Type:	Flowchart
Page Number:	1 of 2
Department/Section:	Hematology
Document Title:	TZANCK TEST

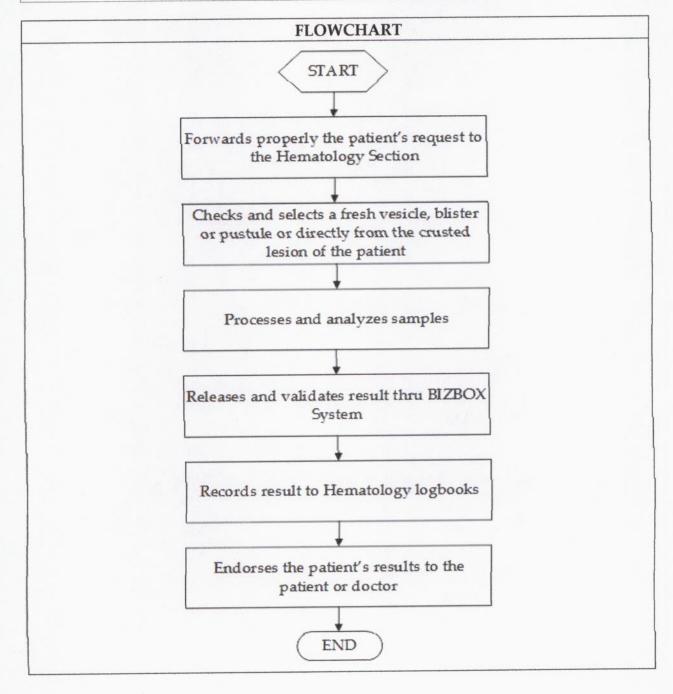



Document Title:	TZANCK TEST	
Departument/Section:	Hermatiology	
Page Number:	2 of 2	
Document Type:	Flowchart	
Effective Date:	06-30-2022	
Document Code:	DPOTMH-E-57-P01-FC25	

# APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	ofaks.	07-06-2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Film	7-6-202
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	dr.llamer	7-6-202
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chief, Department of Pathology	auf - fun	lo 7-6-
Reviewed:	DENNIS C. ESCALONA, MN, FPCHA, FPSQua Quality Assurance Supervisor	0	04-04-2022
Recommending	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer	Relains f.	l. 07.06.20
Approval:	FREDERIC IVAN L. TING, MD OIC-Total Quality Division	B	7/8/2
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO	1	2/11/2