

Document Title:	APT TEST
Department/Section:	Hematology
Page Number:	1 of 3
Document Type:	Standard Operating Procedure
Effective Date:	06-30-2022
Document Code:	DPOTMH-E-57-P01-S14

### **PURPOSE:**

To describe in detail how to prepare and process the APT test correctly and always in the same manner. It is used to differentiate if the blood is from the mother or the neonate.

#### SCOPE:

Applies to all Hematology Section Staff of Laboratory Department of Dr. Pablo O. Torre Memorial Hospital (DPOTMH)

### PERSON RESPONSIBLE:

Medical Technologists (Medical Laboratory Scientists), Pathologists, Medical Doctors, Nurses, Medical Trainees, Laboratory Clerks

#### PROCEDURE:

- 1. Hand washing shall be performed before and after the procedure.
- 2. The wearing of Personal Protective Equipment such as gloves, masks, goggles/face shield and gowns shall be observed in doing the procedure.
- Sample (gastric aspirate) must be placed in a clear, dry, leak-proof, screw-capped container.
- 4. The Medical Technologist or Nurse forwards properly the sample and patient's request to the Hematology Section. Medical Technologist then checks sample label, volume and clots prior to analysis. Not properly labeled sample is given back to the one endorsed for correct labeling.
- 5. Mix 1 part of gastric aspirate to 9 parts of 2% NAOH.

Interpretation of Result/ manner of reporting: <u>PINK</u>: Fetal origin <u>YELLOW OR BROWN</u>: Maternal origin

- 6. The Medical Technologist notes macroscopic findings in the request and encodes results in the system.
- 7. Specimens collected shall be considered as biohazardous material.



Department/Section:  Document Title:	Hematology  APT TEST	
Page Number:	2 of 3	
Document Type:	Standard Operating Procedure	
Effective Date:	06-30-2022	
Document Code:	DPOTMH-E-57-P01-S14	

- 8. The Medical Technologist/Laboratory Clerk prints out results for outpatients at the reception area upon presentation of the Official Receipt (OR) and results for admitted patients can be viewed and printed out by nurses to its respective stations.
- Names and results of patient shall be recorded in the Hematology/Miscellaneous logbook.
- 10. Turn-around time for releasing Apt test result is within 2 hours.

### **REFERENCE:**

https://en.m.wikipedia.org/wiki/Alkali\_denaturation\_test



Document Code:	DPOTMH-E-57-P01-S14
Effective Date:	06-30-2022
Document Type:	Standard Operating Procedure
Page Number:	3 of 3
Department/Section:	Hematology
Document Title:	APT TEST

# APPROVAL:

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	0.44	07-06-2022
	Section Head, Hematology	Luds.	04-04-2022
	TIFFANY B. VILLANUEVA-COO, RMT	Blu	
	Laboratory Manager	<i>pu</i>	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD	A:11	H / 404H
	Laboratory Manager	Arillamers	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP	Warele Center	7-6-2022
	Chair, Department of Pathology	18	
Reviewed:	DENNIS C. ESCALONA, MN, FPSQua	0	07-02-2022
	Quality Assurance Supervisor		01 00 -2072
	ROSARIO D. ABARING, MAN, PhD	(a)1	
Recommending	Ancillary Division Officer	tillaing f. l.	07.06.2022
Approval:	FREDERIC IVAN L. TING, MD	1	7/8/22
• •	OIC - Total Quality Division		,,
Approved:	GENESIS GOLDI D. GOLINGAN	,	
	President and CEO		



Document Code:	DPOTMH-E-57-P01-WI14	
Effective Date:	06-30-2022	
Document Type:	Work Instruction	
Page Number:	1 of 2	
Department/Section:	Hematology	
Document Title:	APT TEST	

	KEY TASKS	PERSON RESPONSIBLE
1.	Encodes the request of the patient to BIZBOX system.	Laboratory Clerks and Nurses
2.	Extracts blood sample from the patients.	
3.	Ensures sample is acceptable for testing.	
4.	Forwards properly the sample and patient's request to the Hematology Section.	
5.	Disposes properly and safely biohazardous and infectious wastes and materials.	Medical Technologist/Nurse
6.	Processes and analyzes samples.	
7.	Releases and validates result thru BIZBOX System.	
8.	Records result to Hematology logbooks.	
9.	Endorses the patient's results to the patient or doctor.	Laboratory Clerks and Nurses




Document Title:	APT TEST	
Department/Section:	Hematology	
Page Number:	2 of 2	
Document Type:	Work Instruction	
Effective Date:	06-30-2022	
Document Code:	DPOTMH-E-57-P01-WI14	

## **APPROVAL:**

	Name/Title	Signature	Date
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT Section Head, Hematology	Luds.	07-04-2022
	TIFFANY B. VILLANUEVA-COO, RMT Laboratory Manager	Plu	7-6-2022
Verified:	MONICA B. VILLANUEVA, RMT, PhD Laboratory Manager	A: llanuer.	7-6-2022
	MELANIE ROSE B. ZERRUDO, MD, FPSP Chair, Department of Pathology	blung-Genedo	7-6-2022
Reviewed:	<b>DENNIS C. ESCALONA, MN, FPSQua</b> Quality Assurance Supervisor	2	07-06-2022
Recommending	ROSARIO D. ABARING, MAN, PhD Ancillary Division Officer	Alaing to 1.	07-06-2022
Approval:	FREDERIC IVAN L. TING, MD OIC - Total Quality Division	13	7/8/22
Approved:	GENESIS GOLDI D. GOLINGAN President and CEO	•	



Document Code:	DPOTMH-E-57-P01-FC14
Effective Date:	06-30-2022
Document Type:	Flowchart
Page Number:	1 of 2
Department/Section:	Hematology
Document Title:	APT TEST

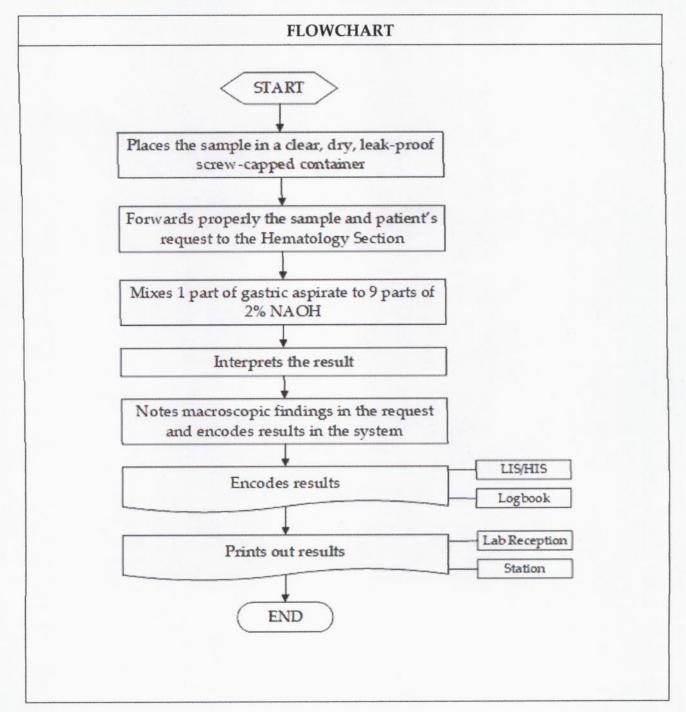



Document Title:	APT TEST
Department/Section:	Hematology
Page Number:	2 of 2
Document Type:	Flowchart
Effective Date:	06-30-2022
Document Code:	DPOTMH-E-57-P01-FC14

# **APPROVAL:**

	Name/Title	Signature	Date	
Prepared by:	REDABELLE DIONEO-SEGOVIA, RMT	-fads.	07 0/ 04-	
	Section Head, Hematology	6-3	07-06-2022	
	TIFFANY B. VILLANUEVA-COO, RMT	Talm	7 / 800 0	
	Laboratory Manager	quu	7-6-2022	
X7 'C' 1	MONICA B. VILLANUEVA, RMT, PhD	A Unmera	7-6-2022	
Verified:	Laboratory Manager	Co. Camura	1-6-2022	
	MELANIE ROSE B. ZERRUDO, MD, FPSP	Lauf funds	7-2-202	
	Chair, Department of Pathology	8	14.70-	
Reviewed by:	DENNIS C. ESCALONA, MN, FPCHA, FPSQua	2	07-04-202	
	Quality Assurance Supervisor			
	ROSARIO D. ABARING, MN, MBA-HA, PhD, FPCHA	01 ,0		
Recommending	Ancillary Services Division Officer	Kelaine 8.4	07.06.202	
Approval:	FREDERIC IVAN L. TING, MD	B	07.06.202	
	OIC- Total Quality Division	1		
Approved:	GENESIS GOLDI D. GOLINGAN			
	President and CEO			